Cavitron® Brand Prophy Powder

DIRECTIONS FOR USE

For Dental use only

ENGLISH

Caution: United States Federal law restricts this device for sale by, or on the order of, a licensed dental professional.

DESCRIPTION: Prophy-Jet® Prophy Powder is a flavored sodium bicarbonate powder. JET-Fresh® Prophy Powder is a flavored aluminum trihydroxide powder. Both prophy powders are for professional air polishing prophylaxis procedures.

NOT FOR INTERNAL USE.

INDICATIONS AND USAGE: For use in conjunction with an air polishing system, for removal of a variety of extrinsic stains, e.g. tobacco, coffee and tea, chlorhexidine. Prophylaxis of orthodontic patients. Preparing tooth surfaces prior to bonding and sealant procedures. JET-Fresh powder is recommended for patients on a sodium restricted diet.

CONTRAINDICATIONS: Patients who have severe respiratory illness should consult their physician before undergoing air polishing prophylaxis procedures. Use is not recommended for patients with known allergies to the components of Cavitron Brand Prophy Powders.

Manufactured by
DENTSPLY Professional
DENTSPLY International

1301 Smile Way
York, PA 17404 USA
Customer Service: 1.800.989.8826

Imported and Distributed in
Canada by
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

DENTSPLY DeTrey GmbH
De-Trey-Str. 1
78467, Konstanz
Germany

WARNINGS: Do not direct the air polishing stream at soft tissue or into the sulcus. Tissue emphysema has been reportedly caused when the air/water/powder stream was directed at the soft tissues or into the sulcus. Do not modify powder with additives. Additives may clog your air polishing system.

PRECAUTIONS: Direct contact of prophy powder with surfaces and marginal areas of dental restorations should be avoided.

ADVERSE REACTIONS: Allergic reactions to components of Cavitron Brand Prophylaxis Powders have been known to occur. If an allergic reaction occurs, seek medical attention immediately.

DIRECTIONS: Shake well before using. Use per polishing unit manufacturer's instructions.

CONTENTS:

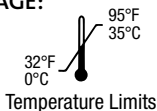
Prophy-Jet Prophylaxis Powder contains Sodium Bicarbonate, Flavoring, Sodium Saccharin.

JET-Fresh Prophylaxis Powder contains Aluminum Trihydroxide and Flavoring.

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